




OMNI-STAT Granules (3g) How to Use/Recommendations for Removal

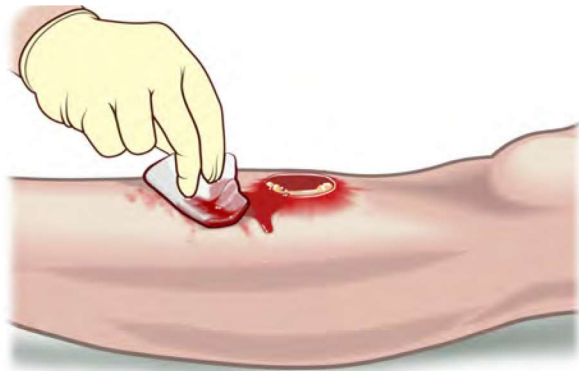


1 Tear open Omni-Stat 3g pouch. Blot away any excess blood or exudate from the wound.

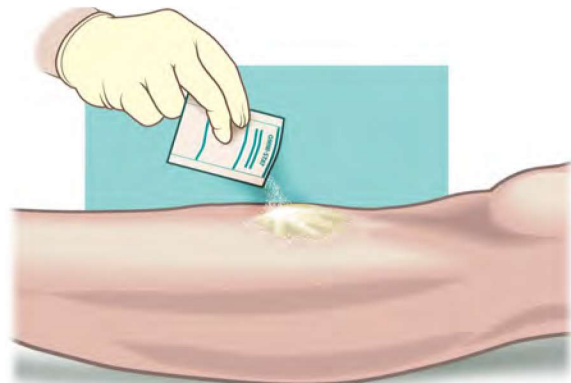


2 Pour enough Omni-Stat onto the wound to cover the entire bleeding area. Omni-Stat should be applied directly to the source of bleeding.

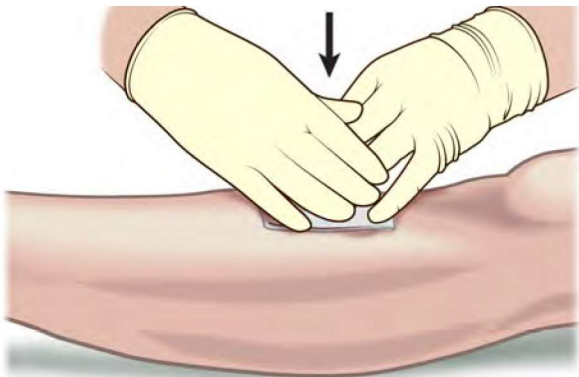

3 Quickly cover with gauze and apply firm pressure for 1 to 3 minutes depending on the severity of the bleeding. If bleeding continues apply pressure for 3 more minutes. If further Omni-Stat is required use an additional pouch.

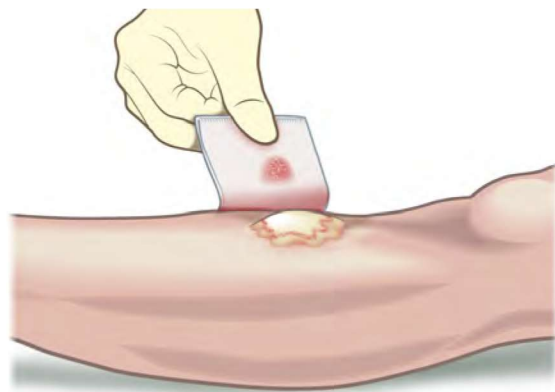

4 Five minutes after bleeding has stopped, gently remove any excess Omni-Stat. Use saline if required. Discard any unused Omni-Stat.

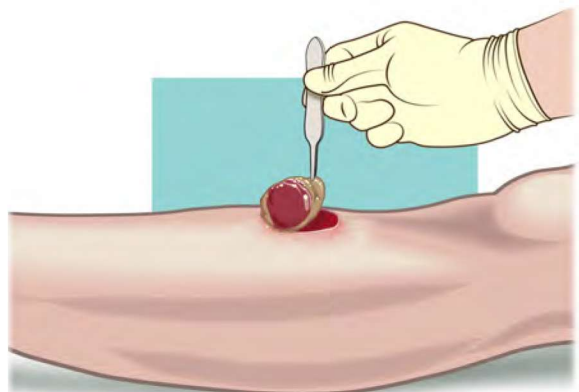

Removal Instructions:

At first dressing change after use, Omni-Stat should be cleansed and removed from the wound using your standard wound cleansing protocols. If required Omni-Stat can be removed (physically) and then any residual irrigated away with water or saline.