


CELOX[™]-A (6g) Granules in Applicator


How to Use/Recommendations for Removal



- 1** Tear open pack, remove applicator while holding pressure on the wound.

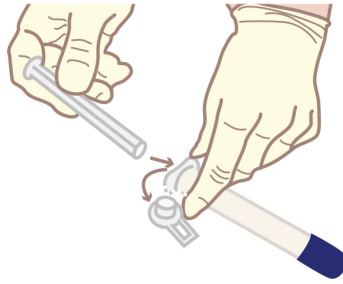

- 2** To prepare the applicator, remove the clear plastic tab from the barrel of the applicator and insert the plunger.

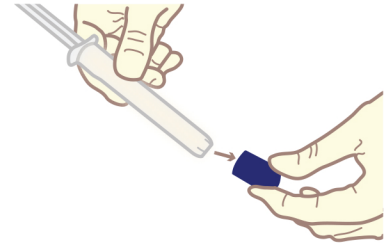

- 3** When ready to apply, remove the blue end cap from the applicator.



- 4** Insert applicator with plunger as far into wound as possible.



- 5** Slowly push the plunger in while withdrawing the applicator and release the granules to fill the wound above skin level. Additional applicators or granules may be used if necessary.



- 6** Cover the wound with standard gauze and apply firm pressure for five minutes or until bleeding stops.



- 7** Secure with a dressing to maintain pressure and discard any remaining unused product.

- 8** If not in a medical facility, seek medical care as soon as possible and show empty pack to medical personnel.

ATTENTION MEDICAL PERSONNEL:

- 1.** Physically remove granules from the wound and any loose surface granules.
- 2.** Fully flood entire wound area with sterile saline irrigation solution.
- 3.** Proceed with normal irrigation and/or suction.
- 4.** Ensure all product is removed from the wound prior to initiation of wound treatment.